For better dentistry

DENTSPLY

AH TempTM

Root Canal Dressing

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Obturation temporaire intra-canalair

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Root Canal Dressing

CAUTION: For dental use only.

Content	Page
1 Product description.....	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Hygiene.....	6
5 Lot number and expiration date.....	7

1 Product description

AH Temp™ root canal dressing is an aqueous, 12.0 pH, calcium hydroxide-based root canal dressing for direct application into the root canal.

AH Temp™ root canal dressing consists of AH Temp™ paste delivered in a syringe and AH Temp™ cannulas for direct placement of the paste into the root canal.

AH Temp™ root canal dressing provides the following clinical advantages:

- Calcium hydroxide is known for its antibacterial effect, stimulates apex fixation and promotes periapical healing.
- The antibacterial properties of the AH Temp™ paste are equal to conventional calcium hydroxide dressings.
- Composition of AH Temp™ paste provides proven compatibility with AH Plus® root canal sealants.
- Reduced desiccation of AH Temp™ paste leads to reduced blocking of the cannula during application and storage.
- Easy navigation through root canal with ultra-thin, long and preangulated AH Temp™ cannula.
- A silicone stop mounted on the cannula allows precise and controlled placement of AH Temp™ paste into the root canal.
- Excellent radiopacity and X-ray visibility compared to leading competitive materials.

1.1 Delivery forms

AH Temp™ paste:

- Luer-Lock syringes containing 0.75 ml paste.

AH Temp™ cannulas:

- Single-use Luer-Lock cannulas, angulation 45°, working length 25 mm, outer diameter 0.41 mm (nominal) (27 gauge), inner diameter 0.31 mm (nominal) with silicone stop.

1.2 Composition

- Calcium hydroxide
- Polyethylene glycol
- Calcium tungstate
- Zirconium oxide

1.3 Indications

- Intermediate, inter-appointment root canal dressing.

1.4 Contraindications

- None known.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains calcium hydroxide which is basic and which may cause burns of skin, eyes and oral mucosa.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If irritation of skin or rash occurs, seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- **Avoid swallowing** to prevent inflammation. If swallowed, immediately rinse out mouth and then drink plenty of water. Seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may reduce the efficacy of the dressing. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use with the original cap.
- Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.
- Deliver paste with AH Temp™ cannulas, staying at least 2 mm short of the apex, always keeping the cannula loose in the canal. Don't deliver with plastic tips which can occlude tight in the canal.
- The AH Temp™ syringe and the finger grip cannot be reprocessed. To prevent the AH Temp™ syringe and finger grip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe and finger grip are handled with clean/disinfected gloves.
- As additional precautionary measure, the AH Temp™ syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier¹.
- Disposa-Shield® protective barrier and AH Temp™ cannulas are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation. Reddish rashes may be seen on the skin.
- Contact with Mucous Membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 28 °C (35 °F and 82 °F). Use the product at room temperature.
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

¹ Disposa-Shield® Infection Control Barrier for syringe-type delivery systems, ReOrder A88013S1.

3 Step-by-step instructions

3.1 Preparation of the root canal

1. Prior to the application of the material prepare, clean, and dry the root canals to be filled using state-of-the-art endodontic techniques.

3.2 Application

Overfilling.

Damage of periapical tissue.

1. Do not place too much material to prevent overfilling of the root canal.
2. If overfilling occurs the material is usually tolerated very well by the surrounding tissue. If however material is pressed into the mandibular canal, immediately apply state of care measures.

Clogging.

Separation of cannula from syringe.

1. Do not allow material to dry inside.
2. Do not reuse steel cannulas.
3. Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.
4. Do not use syringe, once removed from pouch after three months.

The AH Temp™ syringe and the finger grip cannot be reprocessed. To prevent the AH Temp™ syringe and finger grip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe and finger grip are handled with clean/disinfected gloves.

1. Remove cap from the end of the syringe.
2. To assure free flow of material from syringe, express a small amount onto a mixing pad.
3. As an additional precautionary measure, the syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier.
4. Attach disposable cannula over the protective barrier to the end of the syringe.
5. Turn cannula clockwise until fully seated, then tug on cannula to assure locking into syringe collar.
6. Attach finger grip with wings pointing towards front of syringe.
7. To assure free flow of material from the cannula extrude a small amount onto the mixing pad.

Excessive force.

Injury.

1. Apply slow and steady pressure on the syringe.
2. Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.

8. If paste does not flow freely, remove syringe from patient field and check for obstruction.
9. Insert cannula short of the apex. Apply paste slowly withdrawing cannula from the root canal.
10. Remove surplus of paste, place cotton pellets and or temporary restoration to tightly seal the coronal openings of the root canals.
11. Right after use remove finger grip.
12. Push protective barrier half way down the syringe.
13. With index finger and thumb, grasp the longest portion of the protective barrier and pull the protective barrier in a downward motion, over the end of the syringe and cannula, turning the protective barrier inside out.
14. With cannula inside protective barrier, detach the disposable cannula by turning the cannula counter-clockwise.
15. Remove protective barrier without contaminating device and discard protective barrier and cannula according to local regulations.
16. Replace the cap on the syringe head with clean/disinfected gloves.

Tightly sealing of the endodontic cavity using temporary restorative material is essential in order to avoid the contamination of the root canal system between the appointments.

3.3 Removal

To achieve optimal disinfection of the root canal, calcium hydroxide root canal dressings should remain between seven days and four weeks in the root canal.

If used as a long-term root canal dressings, the dressing should be changed every three months.

1. Prior to obturation of the root canal with AH Plus® or AH 26® root canal sealant and root canal points clean AH Temp™ residues from canal walls with root canal preparation instruments and rinse with sterile water.
2. Perform a final rinse with sterile water, dry and obturate.

4 Hygiene

4.1 AH Temp™ syringe and finger grip

Cross-contamination.

Infection.

1. Syringe and finger grip cannot be reprocessed.
2. To prevent the AH Temp™ syringe and finger grip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe and finger grip are handled with clean/disinfected gloves. Do not reuse syringe or finger grip if contaminated.
3. Dispose contaminated syringe and/or finger grip in accordance with local regulations.

4.2 Disposa-Shield® protective barrier

The use of protective barriers is an additional precautionary measure against gross contamination but not against all contamination.

1. Right after use, carefully remove protective barrier together with cannula without contaminating the device.

Cross-contamination.

Infection.

1. Protective barriers are intended for single use only. Do not reuse in order to prevent cross-contamination.
2. Discard in accordance with local regulations.

4.3 Steel cannulas – single use

1. Right after use, carefully remove protective barrier together with cannula without contaminating the device.

Cross-contamination.

Infection.

1. Do not clean, disinfect or reuse steel cannulas.
2. Properly dispose the used steel cannulas in accordance with local regulations.

Clogging.

Separation of cannula from syringe.

1. Do not allow material to dry inside.
2. Do not reuse steel cannulas.
3. Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.
4. Do not use syringe, once removed from pouch after three months.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© DENTSPLY DeTREY 2013-03-08

[These Instructions for Use are based on Master Version 06]

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DENTSPLY DeTrey GmbH
DENTSPLY Limited
DENTSPLY Italia S.r.l.