AutoMatrix® Retainerless Matrix System

For use with Automate® III Tightening Device and AutoMatrix® Snippers+

DIRECTIONS FOR USE – ENGLISH

Caution: For dental use only. USA: Rx only.

1. PRODUCT DESCRIPTION

The AutoMatrix® Retainerless Matrix System is designed to be used as a matrix to contain direct-placement restorative materials in posterior cavities involving proximal surfaces. The AutoMatrix® system allows matrix placement and retention without the need for bulky retainers, allowing for ease of placement, better access, clear view of the operating field and greater patient comfort. Access for wedge placement (where indicated) is not hindered by an external retainer. The single-use, disposable AutoMatrix® system bands allow proper subgingival and proximal contact adaptation while protecting against cross-contamination.

1.1 Delivery forms*

The AutoMatrix® system matrices are available in four height/width and gauge (thickness) configurations (see table below). All AutoMatrix® system matrix sizes fit teeth of any circumference.

Type	Medium Thin ^a	Narrow Regular	Medium Regular	Wide Regular
Height & Gauge	1/4" x .0015	3/16" x .002	1/4" x .002	5/16" x .002
Typical Clinical Uses	Bicuspid/molars	Short Bicuspid and molars	Bicuspid/molars	Long molars

*Some delivery forms may not be available in all countries

^aThin-gauge bands identified by notch on auto-lock loop

1.2 Composition

- Matrix Bands: Polished stainless steel
- Automate® III tightening device: Stainless steel
- AutoMatrix® Snippers+: Stainless Steel, with plastic catch-tray

1.3 Indication

AutoMatrix® Retainerless Matrix System is indicated where placement of a matrix band around a prepared tooth will facilitate contouring of a direct placement restorative material (e.g., Class II preparations).

1.4 Contraindications

None Known.

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

1. AutoMatrix® System matrix band consists of small, sharp parts. As with all intraoral instruments, care must be exercised to ensure stable handling and placement. During matrix removal procedures a segment of the AUTO-LOCK LOOP (retention clip) may separate from the matrix band. Prevent this segment from being swallowed or aspirated. Remove any separated piece immediately from the mouth. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. Wear suitable protective eyewear and gloves.

Aspiration: If aspiration of band or any part occurs seek immediate medical attention.

Ingestion: If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.

Laceration: AutoMatrix® system matrix bands are sharp, and may cut unprotected skin. Handling bands with gloved hands and/or locking college pliers or similar instrument is recommended. In the event a clean band causes a skin laceration, employ usual first aid measures. Discard and properly dispose of band.

2. Used matrix bands and accessories are contaminated. Follow proper exposure control plans. In the event of an accidental exposure, follow recommended post-exposure procedures. Following use of AutoMatrix® system matrix bands, proper handling and disposal techniques are required. Used matrix bands are contaminated and sharp. When handling or disposing, use precautions as when handling or disposing other contaminated sharps.

3. Care should be taken to have good control on any instrument to protect patients from injury. Do not use excessive force when inserting Automate® III tightening device into tightening coil. Do not allow rotating coil to contact oral tissues during tightening. When using removal snippers, employ a technique that prevents entrapping soft tissues (skin, gingiva, mucosa) within the snipper jaws which may result in injury to the tissue. If injury occurs, thoroughly wash the affected area with water and seek appropriate medical attention.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Always make sure that the plastic-basket is properly connected to the AutoMatrix® Snippers+ before application.
- When clipping AUTO-LOCK LOOP with removal snippers, shield open end with fingers to prevent clipped part from projecting into the oral cavity. Immediately retrieve any loose part and remove from the mouth. (See Warnings)
- Avoid placing excessive pressure in an apical direction when placing or on placed bands. Apical force may result in injury to periodontal tissues. (See Warnings)
- Do not use accessory tools other than those designed, supplied, and/or outlined for use with AutoMatrix® system bands. Injury may result. (See Step-by-Step Instructions)

2.3 Adverse reactions

- Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See Contraindications, Warnings and Precautions)
- Failure to properly employ accepted handling and disposal techniques may present biohazard. (See Warnings and Precautions)

2.4. Storage

Store AutoMatrix® system bands and accessory AutoMate® III tightening device and AutoMatrix® Snippers+ at room temperature, away from moisture or excessive humidity. Tightening device and snippers should be disinfected/sterilized and dried prior to storage.

3. STEP-BY-STEP INSTRUCTIONS

3.1 Pre-operative Sterilization

Matrices are intended for single use. If desired, these components may be steam autoclaved one time (134°C, 3 minutes minimum holding time, unwrapped, 12 minutes minimum holding time, wrapped) prior to clinical use. Tightening device and snippers are not supplied sterile, and should be steam autoclaved prior to first and each subsequent use (see Hygiene section).

3.2 Cavity Preparation

Complete cavity preparation as usual utilizing rubber dam or other suitable isolation technique. **Technique Tip:** Pre-wedging, or placement of a Palodent® Plus ring (available separately, see complete Directions for Use) to achieve slight separation may facilitate matrix placement and acceptable restoration proximal contact.

3.3 Matrix selection/preparation-enlarging matrix circumference

Select the appropriate height and gauge matrix band. To properly fit the tooth, matrix should be slightly larger than the occlusal circumference.

3.3.1 Using protected hands, or suitable instrument (e.g. college pliers), unroll the coil of a clean, new AutoMatrix® system matrix band slightly.

3.3.2 While securely holding matrix band (with instrument or by side; do not hold by sharp edges) carefully slide the AUTO-LOCK against the coil. If necessary, repeat steps 3.2.1 and 3.2.2 above to further enlarge matrix to the appropriate size.

3.4. Positioning Matrix

Carefully slide matrix around tooth, through the interproximals. Position matrix with the AUTO-LOCK LOOP at the center of buccal or lingual or most secure remaining tooth wall.

Technique Tips:

1. To seat matrix in tight interproximal spaces – pre wedging, placement of a Palodent® Plus ring or use of an Elliot Separator on posteriors may gain temporarily required space.
2. To clear debris and facilitate seating – use waxed floss or a soft wedge between tight interproximals.
3. For hard to reach molars – use locking college pliers to grip matrix along the side and facilitate matrix seating. Release pliers when matrix is securely seated on tooth.

3.5 Tightening the matrix (using the AutoMate® III tightening device)

3.5.1 Stabilize the matrix with gentle finger pressure apically until the matrix is fully tightened. Check to be sure gingival tissue is not trapped between band and tooth surface. If so, slightly lift and reposition band prior to tightening.

3.5.2 Insert AutoMate® III tightening device tip into the matrix coil. **DO NOT USE FORCE.**

3.5.3 Grasp AutoMate® III tightening device with thumb and forefinger on barrel, palm grip in palm. Rotate knurled barrel clockwise. **NOTE:** AutoMate® III tightening device has an audible torque indicator, thereby indicating the rotational (tightening) force required. After 2 to 3 "clicks", STOP tightening. The band is properly subgingivally tightened. **NOTE:** The AUTO-LOCK LOOP keeps matrix locked tightly until removal is desired.

3.5.4 Remove the tightening device. Rotate AutoMate® III tightening device continuously counterclockwise (backwards) while simultaneously and continuously withdrawing the AutoMate® III tightening device from the coil.

3.6 Placement of restorative

3.6.1 Place wedges as desired (buccal and/or lingual).

3.6.2 Burnish contact areas.

3.6.3 Place desired restorative material as per restorative material manufacturer's directions for use.

3.7 Removing the matrix

Following appropriate restorative curing/setting, open and remove the matrix using shielded AutoMatrix® Snippers+.

3.7.1. Preparation of AutoMatrix® Snippers+

- Align catch-tray with the instrument axis of the snipper.
- Clip Catch-tray straight onto the snippers by sliding the u-shaped cavity into the nut below the bolt head.
- Make sure that the pin on the lower surface of the catch-tray clicks in the hole on the upper side of the blade edge.

Displacement of Catch-tray. Small parts-ingestion/inhalation.

- Do not open Snippers+ more than 60° to prevent displacement of Catch-tray, which may cause leakage of the catch-tray and release of cut pieces.
- Before application, make sure that the catch-tray is properly connected to the snippers: Turn instrument and check that there is no gap between Catch-tray and snippers. Make sure that the catch-tray is empty.

3.7.2 Clip any part of the projecting end of the AUTO-LOCK LOOP.

3.7.3 After clipping immediately retrieve any loose part and remove from the mouth.

3.7.4 Keep snippers closed containing the cut piece during withdrawal from the mouth.

3.7.5 Empty catch-tray and properly discard cut piece.

Small parts-ingestion/inhalation

- Do not open Snippers+ widely during use. Widest opening is designed to dislodge the catch-tray.
- After clipping the AUTO-LOCK LOOP into the catch-tray, keep Snippers+ closed during withdrawal from the mouth.
- If tray is dislodged during use, recover any pieces from the mouth immediately.

3.7.6 Insert an explorer, scaler or suitable instrument through the lock-release hole located on the band near the coil. Gently pull lock-release hole occlusally until the matrix slips open. **NOTE:** Locate the gingival segment of the AUTO-LOCK LOOP. This part will usually remain attached to the band upon removal, but may become dislodged. If segment of AUTO-LOCK LOOP has separated from band, remove it from the mouth and properly discard.

3.7.7 Support the restoration marginal ridge(s) with a condenser. Slide loosened matrix buccally or lingually, while removing matrix occlusally. Properly discard matrix.

4. HYGIENE

Cross-contamination

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Reprocess reusable products as described below.

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Always disassemble Snippers+ catch-tray before final reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).
3. Start reprocessing within 1 hour after use.

For Manual Cleaning and Disinfection

- Discard used gloves according to local regulations.
- Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
- Use a new clean pair of examination gloves.

Instructions for Disinfecting / Sterilizing the Automate® III Tightening Device	
Warnings	<ul style="list-style-type: none"> • High level disinfection by itself is not appropriate for the AutoMate® III tightening device. • Steam autoclaving sterilization is appropriate for the Automate® III Tightening device. • It is not recommended to submerge AutoMate® III tightening device into disinfection solutions. • Do not allow the device to exceed 137° C. • The AutoMatrix® Retainerless Matrix System matrix band is intended for single use only. Do not attempt to reprocess and/or re-use.
Limitations on Reprocessing	<p>The AutoMate® III tightening device can be subjected to the cleaning, disinfection, and sterilization process up to 1000 times. Repeated processing has minimal effect on the AutoMate® III tightening device.</p> <p>Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.</p>
Point of Use	<p>Remove excess soil with disposable cloth / paper wipe.</p> <p>It is recommended that the AutoMate® III tightening device is reprocessed as soon as is reasonably practical following use.</p>
Containment and Transportation	No particular requirements.
Cleaning and Disinfection:	Use only a properly maintained, inspected, calibrated, and approved washer / disinfecter (according to ISO 15883).
Automated	<p>Perform a cleaning program cycle (the following cycle is recommended):</p> <ul style="list-style-type: none"> • 1 minute pre-wash / rinse in cold tap water. • 5 minutes wash with enzymatic detergent at 55° C. • 1 minute enzymatic neutralization in cold tap water. • 1 minute rinse in cold tap water. • Thermal disinfection with minimum A0 of 3000 (e.g. 5 minutes at 90° C). • 20 minutes drying at 100° C. <p>Follow manufacturer's recommendation for use of detergent and neutralizer, observing concentrations and contact times.</p>
Cleaning and Disinfection:	<ul style="list-style-type: none"> • As an alternative to automated cleaning and disinfection, the AutoMate® III tightening device has to be cleaned by scrubbing with hot water and enzymatic detergent solution prepared according to the manufacturer's instructions, e.g., Resurge® Instrument Cleaning Solution. After cleaning thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions. • Remove disinfectant solution residue with a cloth soaked with water. • Dry device with a lint-free single-use cloth
Manual	
Packaging	Paper / plastic steam sterilization pouches may be used, but are not required.
Sterilization	Use a steam autoclave. Do not exceed 137° C. Steam autoclaving - Full cycle: wrapped or unwrapped at 134° C, 2 bar/273°F, 29 psi for 3 minutes.
Drying	Use the drying cycle of the autoclave. Do not exceed 137° C. Allow the Automate® III Tightening Device to fully dry before storage.
Maintenance	<p>If the AutoMate® III tightening device is damaged, worn, or distorted it should be discarded.</p> <p>If the knurled barrel of the AutoMate® III tightening device does not turn freely, add one drop of mineral oil to the ball bearing above the knurling and rotate the barrel to distribute the oil.</p>
Storage	<p>Store the AutoMate® III tightening device at room temperature, away from moisture or excessive humidity. Tightening device should be disinfected/sterilized and dried prior to storage.</p> <p>Instruments steam autoclaved bagged should remain bagged until ready for use. Prior to subsequent re-use, inspect the sterilization pouch and the device. If the integrity of the sterilization pouch has been compromised, the device must be reprocessed prior to use.</p> <p>Instruments steam autoclaved unwrapped should be used immediately.</p>
Manufacturer Contact	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative

Instructions for Disinfecting / Sterilizing the AutoMatrix® Snippers+ and catch-tray	
Warnings	<ul style="list-style-type: none"> • High level disinfection by itself is not appropriate for the AutoMatrix® Snippers+ and catch-tray. • Steam autoclaving sterilization is appropriate for the AutoMatrix® Snippers+ and catch-tray. • Do not allow the device to exceed 137° C. • The AutoMatrix® Retainerless Matrix System matrix band is intended for single use only. Do not attempt to reprocess and/or re-use.
Limitations on Reprocessing	<p>The AutoMatrix® Snippers+ can be subjected to the cleaning, disinfection, and sterilization process up to 800 times.</p> <p>The AutoMatrix® Snippers+ catch-tray can be subjected to the cleaning, disinfection, and sterilization process up to 80 times.</p>
Point of Use	<ol style="list-style-type: none"> 1. For loosening of Catch-Tray, open AutoMatrix® Snippers+ as widely as possible. 2. Pull off Catch-Tray from the AutoMatrix® Snippers+. 3. Clean the devices using a soft brush and running cold water.
Containment and Transportation	No particular requirements.
Cleaning and Disinfection:	Use only properly maintained, calibrated, and approved washer-disinfector ¹ according to ISO 15883-1.
Automated	<ol style="list-style-type: none"> 1. After opening, place AutoMatrix® Snippers+ and catch-tray in the washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during the washer-disinfector-program. Follow washer-disinfector's Instructions for Use. 2. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90°C), using appropriate detergents (e.g.: neodisher® MediClean [0.5%] (alkaline detergent) and neodisher® Z [0.1%] (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany²).

Cleaning and Disinfection:	As an alternative to automated cleaning and disinfection, the AutoMatrix® Snippers+ and catch-tray can also be manually cleaned and disinfected.
Manual	<ol style="list-style-type: none"> 1. Clean AutoMatrix® Snippers+ and catch-tray with a soft brush, cold tap water, and an appropriate cleaning solution (e.g.: alkaline solution: Dürr FD370 [2%]³) containing nonionic surfactants (5 - 15 %), chelating agents (< 5 %) and adjuvants in an aqueous solution, until free of visible contamination. Pay special attention to AutoMatrix® Snippers+ and catch-tray seams and insertions. 2. Remove cleaning solution residue with a cloth soaked with water. 3. Dry AutoMatrix® Snippers+ and catch-tray with a lint-free, single-use cloth. 4. After cleaning thoroughly wipe all surfaces with a single-use cloth in combination with an undiluted, aldehyde-free, alcohol-based, bactericidal, tuberculocidal, virucidal and fungicidal disinfection solution (e.g.: Dürr FD333³) approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions. 5. After 2 minutes remove disinfectant solution residue with a cloth soaked with water. 6. Dry AutoMatrix® Snippers+ and catch-tray with a lint-free single-use cloth.
Packaging	Paper / plastic steam sterilization pouches may be used, but are not required.
Sterilization	<p>After manual cleaning and disinfection or after processing in a washer-disinfector, the AutoMatrix® Snippers+ and catch-tray can also be autoclaved.</p> <ol style="list-style-type: none"> 1. Steam autoclave device unwrapped at 134 °C/ 2 bar = 273 °F/ 29 psi for 3 minutes 30 seconds. Do not exceed 137° C.
Drying	Use the drying cycle of the autoclave. Do not exceed 137° C. Allow the AutoMatrix® Snippers+ and catch-tray to fully dry before storage.
Maintenance	<p>If the AutoMatrix® Snippers+ or catch-tray is damaged, worn, or distorted it should be discarded.</p> <p>Regularly apply medical white oil to the AutoMatrix® Snippers+.</p>
Storage	<p>Store the AutoMatrix® Snippers+ and catch-tray at room temperature, away from moisture or excessive humidity. AutoMatrix® Snippers+ and catch-tray should be disinfected/sterilized and dried prior to storage.</p> <p>Instruments steam autoclaved bagged should remain bagged until ready for use. Prior to subsequent re-use, inspect the sterilization pouch and the device. If the integrity of the sterilization pouch has been compromised, the device must be reprocessed prior to use. Instruments steam autoclaved unwrapped should be used immediately.</p>
Manufacturer Contact	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative

Sterilization of matrices pre-use (if desired) may be accomplished (one time) by steam autoclaving, 134°C, 3 minutes minimum (unwrapped), 12 minutes minimum (wrapped).

5. LOT NUMBER AND EXPECTED USEFUL LIFE

1. AutoMatrix® system accessories are warranted for one year from date of purchase.

2. The following should be quoted in all correspondences:

- Reorder number
- Lot number on the package
- Copy of proof of purchase

Manufactured by
Dentsply Caulk
38 West Clarke Avenue
Milford, DE 19963 USA
Tel.: 1-302-422-4511
www.dentsplysirona.com

Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
Germany
Tel.: 49-7531-583-0
www.dentsplysirona.com

Distributed by
Dentsply Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada

1. For example Miele Washer-Disinfector G7835 CD using program vario TD.

2. Not registered trademarks of Dentsply Sirona, Inc.

3. Not registered trademarks of Dentsply Sirona, Inc.

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©2017 Dentsply Sirona. All Rights Reserved. **563030 WEB** (R 11/3/17)
2 Non sono marchi registrati di Dentsply Sirona, Inc.