SofTip™

DISPOSABLE PROPHY TIPS

• Embouts de prophylaxie jetables • Puntas Prophy desechables • Prophy-
Einwegspitzen • Punte monouso per profilassi • Одноразовые наконечники Prophy

For Single Use Only

Cavitron SofTip Ultrasonic Implant Inserts • Inserts ultrasoniques pour implant Cavitron SofTip •
Inserciones ultrasónicas para implantes Cavitron SofTip • Cavitron SofTip Ultraschall-
Implantateinsätze • Inserti per impianti a ultrasuoni SofTip Cavitron • Ультразвуковые вставки
Cavitron SofTip для имплантатов

DENSPLY
PROFESSIONAL

SofTip Installation Instruction Figures

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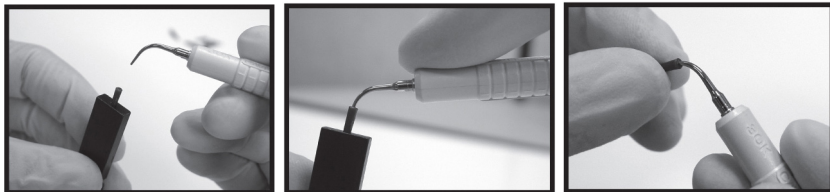
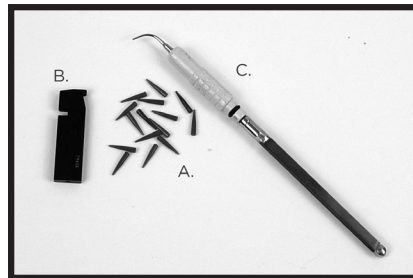
Table of Contents

Indications For Use	5
Contraindications	5
Warnings	5-6
General Precautions	7
Infection Control Procedures	7
Cavitron Ultrasonic Scaler Set-Up.	7
SofTip Disposable Prophy Tip Installation Instructions	8
SofTip Disposable Prophy Tip Removal Instructions	8-9
When To Use The SofTip System	9
Warranty	9

Please read carefully and completely before use.

SofTip Disposable Prophy Tips are designed to work with the Cavitron® SofTip Ultrasonic Implant Insert powered by a 30K Cavitron Ultrasonic Scaler

Indications for Use

The Cavitron SofTip Ultrasonic Implant insert is designed for use only with the SofTip Disposable Prophy Tip and Cavitron ultrasonic scalers for titanium implant and root debridement without damaging effects to the dentine or titanium surfaces.

Contraindications

- The Cavitron SofTip Implant Insert

is not designed, and should not be used, for for primary, hard calculus removal during routine prophylaxis. Primary prophylaxis procedures should be performed using the instruments of your choice (e.g. hand scalers, sonic scaler with standard metal tips, ultrasonic scaler with standard Cavitron ultrasonic inserts).

Warnings

- SofTip disposable prophy tips are not radiopaque. Please observe all warnings and carefully follow directions for use to prevent tip breakage.
- Do not operate the ultrasonic or sonic scaler with the SofTip

Disposable Prophy Tip installed in free air or without water for an extended period as this may cause the disposable prophy tip to break.

- Do not use the SofTip Disposable Prophy Tips with any other ultrasonic insert or sonic scaler tips. The Cavitron SofTip Ultrasonic Implant Insert is designed specifically for use with these disposable tips and use with any other ultrasonic insert or sonic scaler tip may cause the disposable tip to break or dislodge.
- The Cavitron SofTip Ultrasonic Implant Insert is designed for use with Cavitron Ultrasonic Scalers at low power setting.

Use at higher power settings may cause the disposable prophy tip to break.

- Should a disposable prophy tip become dislodged and aspirated by the patient, seek immediate medical attention.
- Do not apply excessive pressure when using the SofTip Disposable Prophy Tip. A light touch assures that the implant fixture will maintain its structural integrity and the implant site will not experience excessive vibrational trauma.

General Precautions

- SofTip Disposable Prophy Tips are intended for single use only. Discard after each use.

Infection Control Procedures

SofTip Disposable Prophy Tips are intended for single use only. Discard after each use. Do not attempt to clean or sterilize them. **DO NOT REUSE THEM.**

The Cavitron SofTip Ultrasonic Implant Insert and the SofTip Wrench should be sterilized after each use following the Infection Control Procedures outlined in the

Infection Control Procedures/ Directions for Use supplied with each insert.

The metal SofTip Base used with the SofTip Wrench should be sterilized after each use following the sterilization instructions provided in the Maintenance section of the Installation, Maintenance and Operation Instructions supplied with the sonic scaler.

Cavitron Ultrasonic Scaler Set-Up

Insert a Cavitron SofTip Ultrasonic Implant Insert (C) into a 30K Cavitron Ultrasonic Scaler handpiece as you would any other

Cavitron insert.

SofTip Disposable Prophylaxis Tip Installation Instructions

Refer to Installation Figures on page 2. Figures show the Cavitron Insert.

1. Take a new plastic SofTip Disposable Prophylaxis Tip (A) and place it, point first, into the hole in the end of the wrench.
2. Ensure the metal tip of the Cavitron Ultrasonic Implant Insert is clean and dry. Insert the metal tip into the opening of the prophylaxis tip and twist 1/4 turn while applying firm pressure to securely seat the disposable prophylaxis tip on the metal tip.

3. Disengage the prophylaxis tip from the wrench. Gently pull on the prophylaxis tip to verify that it is securely in place.
4. Adjust the water flow to obtain a fine spray. The SofTip System is now ready to use.

SofTip Disposable Prophylaxis Tip Removal Instructions

Refer to Removal Figures on page 61. Figures show the Cavitron Insert.

1. Place the plastic SofTip prophylaxis tip in the slot on the SofTip wrench.
2. Place your thumb and index finger over the disposable prophylaxis tip.
3. Gently push the handpiece/sonic scaler downward and the tip will

disengage.

4. Discard the used plastic SofTip Disposable Prophylaxis Tip in accordance with applicable standards for disposal of bio-hazard materials. Proceed with sterilization of the ultrasonic insert or metal SofTip Base and the SofTip wrench.

When to Use the SofTip System

For Implant Maintenance (SofTip Implant Insert)

This instrument is to be used for the removal of plaque and calculus around titanium implants and their superstructures. Use the SofTip

disposable prophylaxis tip by applying light pressure while moving it around the gingival areas of the implants. SofTip disposable prophylaxis tips can also clean metal or porcelain crown and bridgework, as well as screws and screw recesses.

Warranty

For Cavitron SofTip Ultrasonic Insert warranty, refer the Infections Control Procedures/Directions for Use supplied with the insert.

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DENTSPLY DeTrey GmbH