**DELTON® FS+**  
**Light Curing Flowable**  
**Pit and Fissure Sealant**  
**Syringe Delivery System**

**Rx Only. For Dental Use Only.**

DELTON® FS+ provides a quick and simple technique for applying sealant material to the teeth.

DELTON® FS+ is an opaque or tooth colored, 55% glass filled pit and fissure sealant with releasable fluoride.

**COMPOSITION:**

**DELTON® FS+ Pit and Fissure Sealant:**

Low viscosity monomers  
Triethylene glycol dimethacrylate  
BisGMA  
Barium alumino fluoboro silicate glass  
Titanium dioxide (opaque only)  
Sodium Fluoride  
Polymerization initiator  
Stabilizer

**DELTON® EZ Etch Etchant Gel**

Phosphoric Acid, dilute  
Surface active agent  
Silicon dioxide  
Colorant

**INDICATIONS FOR USE:**

Preventive sealing of pits and fissures in the primary and secondary dentition in combination with the acid-etch technique.

**CONTRAINDICATIONS:**

Known allergies to any components.  
Not to be used in case of caries.

**WARNINGS:**

**DELTON FS+ Pit and Fissure Sealant:**

Contains methylacrylates which may be irritating to the skin and eyes. Avoid contact with oral tissues, eyes and skin. If accidental contact with eyes occur, flush affected area with generous amounts of water and seek medical attention. If accidental contact with skin occurs, wash immediately with plenty of soap and water. The product may cause sensitization by skin contact in susceptible persons. If skin sensitization occurs discontinue use.

**DELTON® EZ Etch Etchant Gel:**

Contain phosphoric acid which may be irritating to the skin and eyes. Avoid contact with oral tissues, eyes and skin. If accidental contact with eyes occur, flush affected area with generous amounts of water and seek medical attention. If accidental contact with skin occurs, wash immediately with plenty of soap and water.

**PRECAUTIONS:**

DELTON® FS+ Pit and Fissure Sealant and DELTON® EZ Etch Etchant Gel syringes should be tightly closed immediately after use. Keep DELTON® FS+ Pit and Fissure Sealant and DELTON® EZ Etch Gel out of direct sunlight. Recommended storage; do not exceed 24° C (75° F). Recommended usage; use at room temperature.

**ADVERSE REACTIONS:**

Since reversible inflammatory changes of the oral mucosa may occur after accidental contact with acrylate monomers, avoid contact of Delton® with mucous membranes.  
Anaphylactic reactions may occur in susceptible individuals. If patient shows signs of anaphylaxis, seek medical attention immediately.

**STEP-BY-STEP INSTRUCTIONS:**

**Cleaning:**

Clean surfaces to be sealed with an air-prophylaxis device (Cavitrone® Prophyl-Jet or Cavitrone® JET Plus) or an oil-free paste (NUPRO® Prophyl Paste) and a disposable prophyl angle (NUPRO® revolv®). Rinse well with water.

**Isolating:**

Isolate the teeth to be sealed with a rubber dam or cotton rolls and dry each tooth with air, free of oil or water contamination.

**Etching:**

**Application of DELTON® EZ Etch Etchant Gel:**

**NOTE:** Apply a disposable barrier (Disposa-Shield®) to syringe to reduce cross-contamination between patients and dental practitioners. See Directions For Use for Disposa-Shield® Disposable Syringe Sleeve. Remove the cap from the DELTON® EZ Etch syringe. Attach a disposable brush tip applicator to end of the syringe. Turn tip clockwise ¼ to ½ turn to assure that it is fully seated. Tug on the tip to be sure that it is locked into the collar of the syringe. To assure free flow of material from syringe, express a small amount onto a pad. Apply etchant into the pits and fissures through brush tip applicator. Etch each tooth for 30 to 60 seconds. Rinse teeth thoroughly for at least 30 seconds each. If cotton rolls are used, they should be changed without contaminating the treated surfaces. Dry conditioned areas with air, free of oil or water contamination. Properly etched enamel has a dull frosty-white appearance. If this is not the case, re-etch enamel for another 20 seconds. Once the surfaces have been properly treated, they must be kept uncontaminated. If salivary contamination occurs, thoroughly clean with forceful water spray and dry.

**Sealing:**

**Use at room temperature.**

**Application of DELTON® FS+ Pit and Fissure Sealant:**

**NOTE:** Apply a disposable barrier (Disposa-Shield®) to syringe to reduce cross-contamination between patients and dental practitioners. See Directions For Use for Disposa-Shield® Disposable Syringe Sleeve.

Remove the cap from the DELTON® FS+ syringe. Attach a disposable brush tip applicator to end of the syringe. Turn tip clockwise ¼ to ½ turn to assure that it is fully seated. Tug on the tip to be sure that it is locked into the collar of the syringe. To assure free flow of material from syringe, express a small amount onto a pad. Apply sealant into the pits and fissures through brush tip applicator.

**Curing:**

Cure each surface for at least 20 seconds with a polymerization unit keeping the light exit window as near as possible to the tooth without touching it. Remove soft (oxygen-inhibited) surface layer after light-curing with cotton pellets or cotton rolls.

**Retention and Occlusion Control:**

Check sealed area with an explorer for complete coverage and retention. Check occlusion with articulating paper and adjust if necessary with a finishing bur.

**Syringe Care:**

**DELTON® FS+ Pit and Fissure Sealant syringe and DELTON® EZ Etch Etchant Gel syringe care.**

**NOTE:** Remove disposable barrier (Disposa-Shield®) from syringe. See Directions For Use for Disposa-Shield Disposable Syringe Sleeve. Carefully remove the spent brush tip applicator from the syringe and discard appropriately. Replace each cap tightly. Wipe the entire syringe with disinfectant and dry with clean cloth.

**STORAGE:**

Syringes should be closed immediately after use. Keep out of direct sunlight, do not store at temperatures exceeding 24° C (75° F).

**BATCH NUMBER AND EXPIRY DATE**

Do not use after expiry date.  
The batch number indicates important production information. The number should be quoted in all correspondence which requires identification of the product.

Manufactured by:  
DENTSPLY Professional  
DENTSPLY International  
York, PA 17404  
Tel: 1-800-989-8826

Distributed by  
DENTSPLY Canada  
Woodbridge, Ontario  
L4L 4A3

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