**NUPRO**

**Freedom®**

**DISPOSABLE PROPHY ANGLE**

For MIDWEST® RDH Freedom® Cordless Prophylaxis System

*with latex free soft cup*

### Indications for Use

Nupro Freedom® disposable prophylaxis angle is designed for single use only as part of a professionally administered prophylaxis treatment. This angle is designed to be compatible with the Midwest® RDH Freedom® Cordless Prophylaxis Handpiece.

### Contraindications

None known.

### Warnings

Dispose of Nupro Freedom® Disposable Prophylaxis Angles after each patient according to CDC Guidelines for Infectious Waste and Federal, State and Local regulations.

### Precautions

- The Disposable Prophylaxis Angles are designed for single-patient use only and should never be used more than once. Disposable Prophylaxis Angles are not autoclavable or designed to withstand disinfection solutions.
- The risk of reuse of a Disposable Prophylaxis Angle are damage to equipment and cross-contamination. Install a new Prophylaxis Angle before each use.

### Adverse Reactions

None known.

### Directions for Use

1. Remove the Nupro Freedom® disposable prophylaxis angle from the package by tearing open at the notches.
2. Attach a Nupro Freedom® Disposable Prophylaxis Angle to the handpiece by aligning the swoops on each and snapping in place. Verify that all parts of the handpiece are securely attached before use.
3. Dispose of the Nupro Freedom® disposable prophylaxis angle after each patient according to CDC Guidelines for Infectious Waste and Federal, State and Local regulations.

CE

0086

Manufactured for:  
DENTSPLY Professional  
DENTSPLY International  
1301 Smile Way  
York, PA 17404  
Customer Service:  
(800) 989-8826

Imported and Distributed  
in Canada by  
DENTSPLY Canada  
Woodbridge, Ontario  
L4L 4A3

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