# Rinn® XCP® XCP-ORA® XCP-DS® XCP-DS FIT®

Holders and Positioners for Intraoral Radiography

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STEAM  
AUTOCLAVE

Manufactured by  
DENTSPLY RINN  
1301 Smile Way  
York, PA 17404 USA  
Dentsply.com

Customer Service: 800-323-0970

DENTSPLY Limited  
Building 3, The Heights,  
Weybridge, Surrey KT13 0NY UK

**Caution:** U.S. Federal laws restrict this device to sale to or on the order of a licensed dental professional.

**Indications for use:** XCP® holders are indicated for holding film and phosphor plates for intraoral dental radiographs. XCP® and XCP-ORA® arms and rings are indicated for positioning and aligning dental intraoral radiographs. XCP-DS® and XCP-DS FIT™ holders are indicated for holding digital sensors for intraoral dental radiographs.

**Contraindications:** None known.

**Warnings:** Sterilize before use. DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemically, dry-heat sterilize or cold sterilize. Only sterilize with steam autoclaving according to cleaning and sterilization instructions. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

**Precautions:** Follow sterilization instructions carefully to prevent plastic damage during sterilization. When using with corded digital sensors, use a sensor cover and do not pull on cord during use.

**Adverse Reactions:** None known.

#### Step-by-Step Instructions:

1. Insert positioning arm into the appropriate color-coded channel of the aiming ring.
2. Attach appropriate biteblock for area to be imaged. Correctly assembled, the imaging area should be centered when viewed through the aiming ring.
3. Position holder in patient's mouth and slide ring close to patient's face.
4. Align x-ray tube to ring.
5. Take exposure.
6. Adjust as needed for the next desired position.

**Reprocessing:** Disassemble instrument(s) and separate all parts prior to reprocessing.

#### Manual Cleaning and Disinfection:

1. Rinse components under running tap water using a soft-bristled brush to remove gross soil using an enzymatic detergent per manufacturer's recommendation, paying special attention to any cavities or holes.
2. Rinse under running tap water and dry with cloth or disposable wipe.
3. After thoroughly cleaning, wipe all surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (such as VoloWipe) following manufacturer's instructions. Pay special attention to any cavities or holes.
4. Rinse components by wiping them with a cloth soaked in pure water or by running under running tap water.

#### Automated Cleaning and Disinfection:

As an alternative to manual cleaning and disinfection, the devices can also be processed in a washer-disinfector. Use only properly maintained, calibrated, and approved washer-disinfectors according to ISO 15883-1.

1. Place devices in washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during washer-disinfector program following manufacturer's instructions. Run a washer disinfector program with A0 value  $\geq$  600.

#### Sterilization:

1. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132°C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 273°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

**Storage:** Store at room temperature (18-25°C / 65-77°F).

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