Sani-Tip®

C-1 / C-2 / C-3 Conversion Kits:

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Rx Only.

INDICATIONS FOR USE

Used to transmit air or water to the operatory field during operative procedures.

CONTRAINDICATIONS



This Conversion Kit is not indicated for use with any other air/water syringes.

WARNINGS

- Sani-Tip® Conversion Kits are designed to function exclusively with Sani-Tip® disposable syringe tips. Use only adapter components and clear Sani-Tip plastic syringe tips manufactured by DENTSPLY Professional for the Sani-Tip® System to ensure optimal performance and safety including reducing potential for cross-contamination. Follow the enclosed instructions.
- Sani-Tip® disposable syringe tips are for single use only. Dispose after use.
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- To ensure adequate protection from cross contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti-retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer.
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used.


PRECAUTIONS

- Although this conversion kit may be used with all Sani-Tip disposable syringe tips, it is recommended for use with the mini (57 mm) Sani-tip disposable syringe tips.
- To help reduce cross contamination, use a new Sani-Shield disposable air/water syringe sleeve, in addition to a new Sani-Tip disposable tip, for every patient.

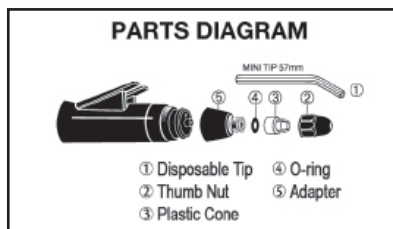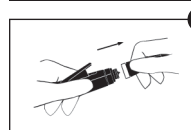C-1 CONVERSION KIT FOR CLEANTHREE & CLEANWARM AIR/WATER SYRINGES**C-1 CONVERSION KIT ADAPTER INSTALLATION INSTRUCTIONS**

1


1. Remove existing syringe tip assembly. Check condition of o-rings on syringe head. If damaged or missing, contact Castellini to replace.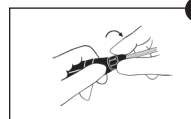

2


2. Hand-thread new adapter assembly onto syringe. Do not over-tighten.

Installation is complete.

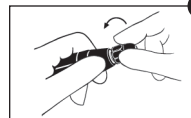C-1 CONVERSION KIT USER INSTRUCTIONS


1

1. Hold adapter securely to prevent unthreading of adapter. With other hand, loosen thumb nut.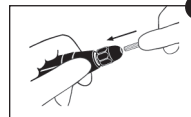

2

2. With thumb nut loosened, insert new Sani-Tip® into adaptor as far as possible. While holding adaptor body and pushing in on tip, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.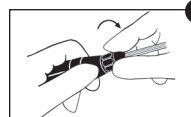

3

3. Hand-tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

TEST PRIOR TO EACH USE: Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.

4. To replace tip, hold adapter body and loosen thumb nut. Remove and discard used tip. Repeat steps 2 and 3.

INFECTION CONTROL PROCEDURES

WARNINGS

- Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

LIMITATIONS ON PROCESSING

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

INSTRUCTIONS

Point of use:

Remove excess soil with disposable cloth or paper.

Containment and Transportation:

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

Preparation for decontamination:

Remove excess debris from the components using a disposable cloth or paper. Discard cloth/paper after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter o-ring.

Cleaning: automated

None.

Cleaning: manual

1. Clean all external surfaces of the Sani-Tip® conversion kit, including the air/water syringe with a mild detergent and water solution. Dry with a lint-free cloth.
2. Remove the conversion kit from the air/water syringe.
3. Clean all internal surfaces of the adapter, thumb nut, plastic cone and o-ring with a mild detergent and water solution. Dry with a lint-free cloth.
4. Further process the adapter and thumb nut following the Sterilization procedure listed.
5. The plastic cone and o-ring are not steam autoclave sterilizable. Further process following the disinfection procedures listed.

Disinfection:

Disinfect the plastic cone and o-ring using standard disinfection solutions.

Packaging:

Place thumb nut and adapter in a paper or paper/plastic steam sterilization pouch. Components may be loaded into dedicated instruments tray.

Sterilization:

Use a steam autoclave. Place bagged adapter and thumb nut into the steam autoclave, paper side up

when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa/31psi for 3 to 12 minutes.

Alternate method: Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa /31psi for 3 to 4 minutes. Components should be used immediately.

Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

Maintenance, inspection and testing:

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

Storage:

To maintain sterility, instruments should remain bagged until ready to use.

MANUFACTURER CONTACT

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Representative.

C-1 TROUBLESHOOTING

PROBLEM: Water leaking, wet air, loose tips, and/or loose adapter, indicates either:

- Center hole of Sani-Tip® disposable tip is not fully seated over water spindle inside new adapter.
- Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; existing o-rings on syringe head are missing or damaged; thumb nut has not been fully tightened, or incorrect tips are being used.
- Adapter has been inadvertently unthreaded.
- Internal water spindle is damaged.

SOLUTION:

- Loosen thumb nut before interchanging Sani-Tip® disposable tips. Insert new tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a water-tight seal over internal tapered water spindle. Tighten thumb nut.
- Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all o-rings. If existing o-rings on syringe head are damaged or missing, contact Castellini to replace. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- When loosening thumb nut, hold adapter to prevent inadvertent unthreading of adapter.
- If internal water spindle inside Sani-Tip® adapter is damaged, replace adapter.

PROBLEM: Crossover of water droplets into the air line caused by poor hose connection or worn valve o-rings at base of syringe.

SOLUTION: Confirm hose connection at base of syringe is secure. Replace worn or missing valve o-rings.

PROBLEM: Tip does not fit through thumb nut.

SOLUTION: Loosen thumb nut a full turn to expand opening of plastic cone.

PROBLEM: Little or no water.

SOLUTION: Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.


PROBLEM: Too much/little air or water pressure.

SOLUTION: Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

ALWAYS TEST PRIOR TO USE:


Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.

C-2 CONVERSION KIT FOR MULTISTERIL AIR/WATER SYRINGES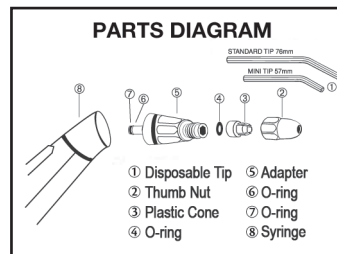

C-2 CONVERSION KIT ADAPTER INSTALLATION INSTRUCTIONS

- Please read the following instructions carefully before installing the Sani-Tip Conversion Kit. Failure to follow instructions could result in damage to your air/water syringe.

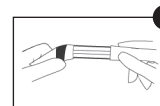
- ull to remove internal assembly from syringe sleeve.

- Pull tip to create small opening between tip and syringe sleeve. Then insert removal tool into small opening.


- Pull tool toward syringe to loosen tip for easy removal.
- Remove existing syringe tip (pull off).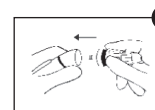

- Verify adapter has 3 o-rings.


- Firmly push Sani-Tip adapter as far as possible into syringe opening. Adapter will snap audibly when fully engaged and will be flush with syringe base. Failure to fully insert adapter will result in leakage and/or loose adapter.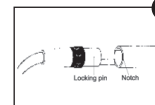


- Line up locking pin on internal assembly with notch on the syringe sleeve, push together until firmly seated.

Installation is complete.


C-2 CONVERSION KIT USER INSTRUCTIONS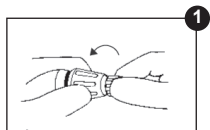

1. Hold syringe and adapter securely to prevent rotation. With other hand, loosen thumb nut.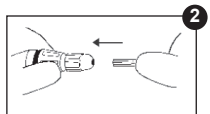

2. With thumb nut loosened, insert new Sani-Tip® disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.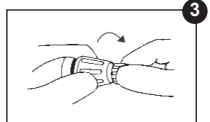

3. Continue to hold the syringe, adapter and hand-tighten thumb nut. To redirect tip, rotate adapter rim.

TEST PRIOR TO EACH USE: Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.

4. To replace tip, hold syringe, adapter body and loosen thumb nut. Remove and discard used tip. Repeat steps 2 and 3.

C-2 TROUBLESHOOTING

PROBLEM: Water leaking, wet air, and/or loose adapter, indicates either:

- a.) Center hole of Sani-Tip® disposable tip is not fully seated over water spindle inside new adapter.
- b.) Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c.) Adapter is not fully mounted onto syringe.
- d.) Internal water spindle is damaged.

SOLUTION:

- a.) Loosen thumb nut before interchanging Sani-Tip® disposable tips. Insert new tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a water-tight seal over internal tapered water spindle. Tighten thumb nut.
- b.) Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all o-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- c.) Firmly push adapter as far as possible onto syringe. Adapter must be flush with syringe face to make proper connection.
- d.) If internal water spindle inside Sani-Tip® adapter is damaged, replace adapter.

C-2 TROUBLESHOOTING continued

PROBLEM: Tip does not fit through thumb nut.

SOLUTION: Loosen thumb nut a full turn to expand opening of plastic cone.

PROBLEM: Little or no water.

SOLUTION: Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.


PROBLEM: Too much/little air or water pressure.

SOLUTION: Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.


ALWAYS TEST PRIOR TO USE:

Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.


C-3 CONVERSION KIT FOR MULTISTERIL & THREESTERIL AIR/WATER SYRINGES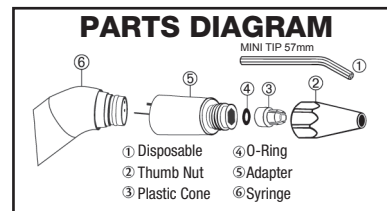


C-3 CONVERSION KIT ADAPTER INSTALLATION INSTRUCTIONS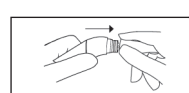

1. Remove existing syringe tip assembly (pull off).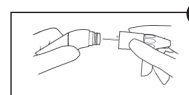


2. Insert the longest metal tube on the Sani-Tip adapter into the smallest hole opening of the syringe. Next, align and insert the shortest metal tube on the Sani-Tip adapter with the larger hole opening of the syringe.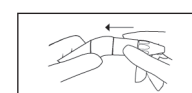


3. Firmly push Sani-Tip adapter as far as possible into syringe opening. Adapter will snap when fully engaged and adapter base should be flush with syringe base. Adapter will fit snugly over syringe opening. Failure to fully insert adapter will result in leakage and/or loose adapter.

Installation is complete.


C-3 CONVERSION KIT USER INSTRUCTIONS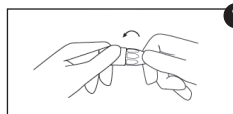

1. Hold syringe and adapter securely to prevent rotation. With other hand, loosen thumb nut (counter-clockwise).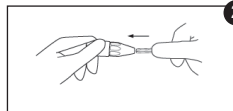

2. With thumb nut loosened, insert new Sani-Tip® disposable tip and push into adapter as far as possible. While holding adapter body and pushing in on tip, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.

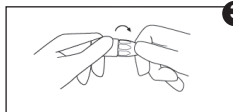3. Continue to hold the syringe and adapter and hand-tighten thumb nut. To redirect tip, rotate adapter rim.
4. To replace tip, hold syringe and adapter and loosen thumb nut. Remove and discard used tip. Repeat steps 2 and 3.

TEST PRIOR TO EACH USE: Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.

C-3 TROUBLESHOOTING

PROBLEM: Water leaking, wet air, loose tips and/or loose adapter, indicates either:

- a.) Center hole of Sani-Tip® disposable tip is not fully seated over water spindle inside new adapter.
- b.) Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c.) Adapter is not fully mounted onto syringe.
- d.) Internal water spindle is damaged.

SOLUTION:

- a.) Hold adapter to prevent rotation and loosen thumb nut before interchanging Sani-Tip® disposable tips. Insert new tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut.
- b.) Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all o-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- c.) Firmly push adapter as far as possible onto syringe. Adapter rim must be flush with syringe face to make proper connection.
- d.) If internal water spindle inside Sani-Tip® adapter is damaged, replace adapter.

C-3 TROUBLESHOOTING continued

PROBLEM: Tip does not fit through thumb nut.

SOLUTION: Hold adapter to prevent rotation. Loosen thumb nut a full turn to expand opening of plastic cone.

PROBLEM: Little or no water.

SOLUTION: Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.

PROBLEM: Too much/little air or water pressure.

SOLUTION: Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

PROBLEM: Adapter leaks.

SOLUTION: Replace existing o-rings on syringe.

ALWAYS TEST PRIOR TO USE:

Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.

INFECTION CONTROL PROCEDURES

WARNINGS

- Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

LIMITATIONS ON PROCESSING

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

INSTRUCTIONS

Point of use:

Remove excess soil with disposable cloth or paper.

Containment and Transportation:

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

Preparation for decontamination:

Remove excess debris from the components using a disposable cloth or paper. Discard cloth/paper after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter o-ring.

INFECTIION CONTROL PROCEDURES

INSTRUCTIONS continued

Cleaning: automated
None.

Cleaning: manual

1. Clean all external surfaces of the Sani-Tip® conversion kit, including the air/water syringe with a mild detergent and water solution. Dry with a lint-free cloth.
2. Remove the conversion kit from the air/water syringe.
3. Clean all internal surfaces of the adapter, thumb nut, plastic cone and o-ring with a mild detergent and water solution. Dry with a lint-free cloth.
4. Further process the adapter and thumb nut following the Sterilization procedure listed.
5. The plastic cone and o-ring are not steam autoclave sterilizable. Further process following the disinfection procedures listed.

Disinfection:

Disinfect the plastic cone and o-ring using standard disinfection solutions.

Packaging:

Place thumb nut and adapter in a paper or paper/plastic steam sterilization pouch. Components may be loaded into dedicated instruments tray.

Sterilization:

Use a steam autoclave. Place bagged adapter and thumb nut into the steam autoclave, paper side up when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa/31psi for 3 to 12 minutes.

Alternate method: Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa /31psi for 3 to 4 minutes. Components should be used immediately.

Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

Maintenance, inspection and testing:

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

Storage:

To maintain sterility, instruments should remain bagged until ready to use.

MANUFACTURER CONTACT

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Representative.

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The enclosed adapter is covered, in whole or in combination with other parts of the SANI-TIP® System, by one or more of the following patents: U.S. 5,306,146; U.S. 5,489,205; AU 714134; CA 2,155,313; EP 0686017.

You do not have a license, express or implied, under any of these patents to make or sell the adapter. You have an express license only under Patent Numbers U.S. 5,306,146; and U.S. 5,489,205; and no implied license under any other of these patents, to use the adapter in combination with the enclosed syringe tip(s) furnished by DENTSPLY Professional.

If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you are in violation of this license granted to you and you may be liable for willful patent infringement. If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you do so entirely at your own risk and DENTSPLY Professional denies any responsibility for such use, including product liability and any express or implied warranties.

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DENTSPLY DeTrey GmbH